



Biomedical Polymers, Inc.

Hemodrop Blood Dispenser

*An easier, safer way
to prepare differentials*

The Hemodrop blood dispenser is designed for the safe and easy dispensing of a single drop of blood from any style of specimen tube without removing the cap. By eliminating the need to remove the cap from the specimen tube, the introduction of airborne pathogens is minimized.



Features and Benefits

- ▶ **Save time.** No need to uncap specimen tubes to make slides, so you can make smears faster.
- ▶ **Easy to use.** The flat surface of the Hemodrop makes insertion into the tube easy and painless.
- ▶ **Visibility.** The clear plastic of Hemodrop allows you to see the drop on the slide so you know when your slide is ready.
- ▶ **Safe.** Hemodrop is disposable, helping to avoid any unnecessary hazards.

How to Use

Making differentials has never been easier. Three easy steps are all it takes for the easy dispensing of a drop of blood from a specimen tube onto a slide.

1. Insert the Hemodrop blood dispenser into the specimen tube with your thumb.
2. Turn the tube upside down and gently press down on the slide until a uniform drop of blood appears.
3. Once drop of blood appears, relax pressure and carefully lift the specimen tube from the slide.

Ordering Information

| Part Number | Description | Quantity |
|--------------|--------------------------|-----------|
| BMP-HEMODROP | Hemodrop blood dispenser | 1000/case |

42 Linus Allain Avenue • Gardner, MA 01440 USA
 Toll Free: 800/253-3684 • Tel: 978/632-2555
 Fax: 978/632-2524 • E-mail: sales@biomedicalpolymers.com
 www.biomedicalpolymers.com